


LabTest XM

Canine procedure for major crossmatch

Material provided:

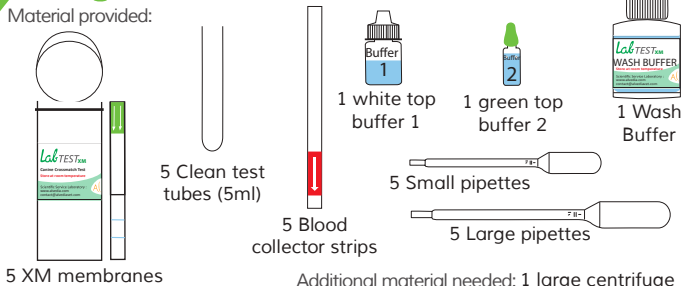

N°1: PREPARATION OF BLOOD SAMPLES


(MAJOR = Donor RBC + Recipient serum or plasma)

Recipient

Centrifuge blood tube in order to collect :

Whole blood



- EDTA
 - ACD
 - CPD
 - Plain tube
- Do not use Heparin

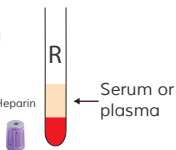


Donor


Centrifuge blood tube. Discard the serum or plasma to collect pRBCs.

Collect blood from blood bag segment.

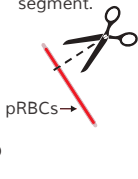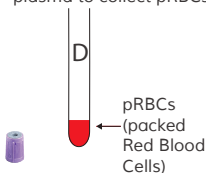


N°2: XM TEST PROCEDURE

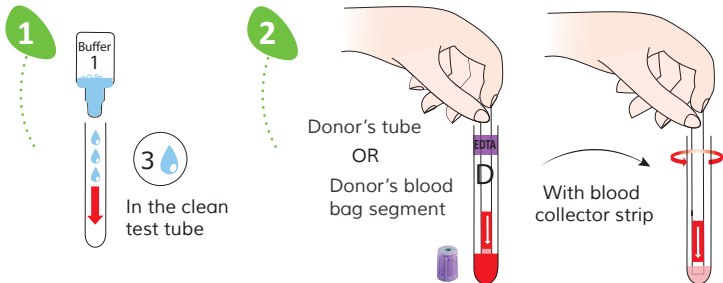

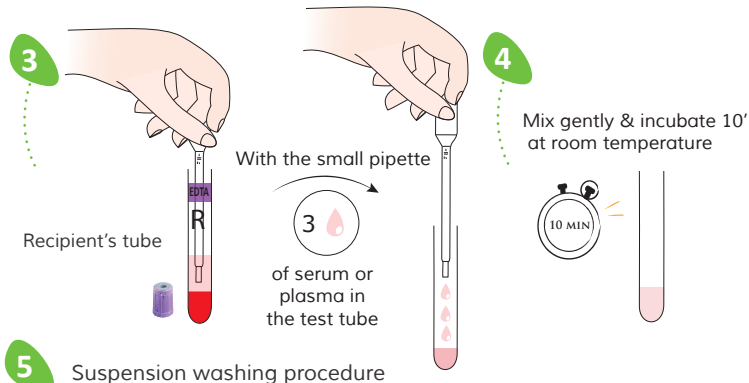

5 Suspension washing procedure

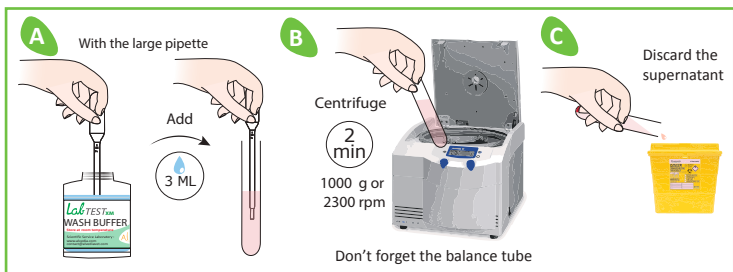

Suspension washing procedure (x3) **STEP 5 MUST BE DONE 3 TIMES**

